

ON CAMPUS



Lifeloc Breath Alcohol Testers

Lifeloc Breath Alcohol Testers effectively identify student alcohol consumption and deter alcohol abuse at school events. Rapid testing of students and open drink containers keeps students safe and make events more enjoyable for all.

"I am convinced that because of our Lifeloc breathalyzer, the number of students who chose to use alcohol while in attendance at our winter formal and our prom was next to none."

High School Principal, California

Passive Testing

The FC5 Hornet alcohol tester is designed for zero tolerance alcohol screening. Three test modes give you ultimate control and flexibility in the fast, easy and reliable testing of students. No mouthpieces or training courses required. Test students, ambient air and even open drink containers.

The FC5 Hornet is designed for automatic or manual alcohol screening. In auto mode simply have the subject blow onto the collector cone and the test will trigger automatically. A POS (alcohol detected) or NEG (no alcohol) test result will appear almost instantly. Perfect for the rapid screening of large groups. In manual mode you decide when to trigger if testing ambient air or open drink containers.

School alcohol screening made simple, reliable and affordable.



Lifeloc
TECHNOLOGIES
Unlock the Power of Alcohol Testing



Ease of Use


- Icon driven software
 - Easily guides users through the testing process
- LCD screen with visual cues
 - Easy to interpret
 - Backlit for day/night use
 - Battery indicator
- Audible alerts

Precision Instrument

- Rapid & repeatable test taking
 - 100's of tests in a short period
 - Instantaneous negative result reading and <30 for a positive result
- Double pump action ensures highly accurate test sample in open container mode
- Wide operating temperature range
 - 0-55° C or 32-130° F


Test Mode

POS/NEG

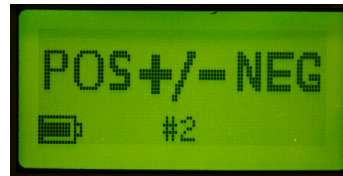


- Test auto triggered when student blows enough air toward collector cone
- Manual Override for uncooperative student
- Instant +Pos or -Neg result

Zero/Low/High

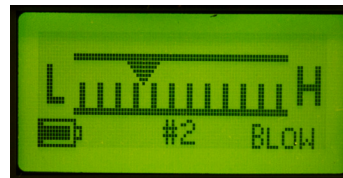

- Alerts operator to one of three levels of alcohol
 - Zero: .000-.010
 - Low: .011-.020
 - High: .021+

Open Container

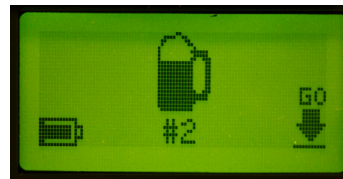

- Samples air-space over open containers and spills
- One button manual trigger



"... we were getting a lot of incidences happening at schools during prom, homecoming and different high school dances. The school checked over 450 students within 30 minutes without a problem. This Friday is the second prom...I believe that there won't be any problems."

School Administrator, Louisiana

www.lifeloc.com/schools.aspx

Lifeloc FC5 Hornet

Professional Breath Alcohol Tester